

FAQ – BLUETOOTH TYTRON

How do you install the Tytron on your computer?

Make sure the scanner is charged and **NOT** plugged in.

Go to Device Manager to see if your computer has Bluetooth built in.

You can get to the Device Manager from the Scanner tab in Setup.

If Bluetooth **IS NOT built in** to computer

Connect Bluetooth dongle, wait for drivers to load

Follow instructions as if Bluetooth is built in

If Bluetooth **IS built in**

Go to Devices and Printers

Select Add device, select C7000-7XXXX

Use pairing code 2728

Wait for drivers to finish installing

Right click on C7000 and select Properties

Go to Services tab to see which port it is using

Go into the Tytron Software and select Setup

Go to Scanner tab and enable Serial Scanner 1 and enter the port number

Then plug in the scanner and use the Copy Calibration button to set the calibrations

How do I charge the Tytron?

To charge the Blue Tooth scanner, you will use the provided USB cord to the wall outlet. The scanner uses a cell phone battery, so you will want to charge it every night.

What do the lights on the scanner mean?

If the light on top is blinking it is ready to connect. A solid light means it is connected. If the light is off the battery is dead or it is plugged in.

Can I use the scanner on networked computers in the office?

It can connect to multiple computers. Once the patient is unloaded the light should start flashing meaning it's ready to connect to a different computer. When you load a patient the light should go solid showing the connection. Once it is connected it can't connect to another until you disconnect by unloading the patient.

What is the range of the Bluetooth link?

Bluetooth has a range of 10 meters barring interference from furniture, walls, people and etc.